KÄYTTÖOHJE

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Suojaimen merkki: Lumi Medical

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Disposable Medical Face Mask

Brand: Lumi Medical

Product Description: 3-Layer disposable medical face mask, earloops

Mask Type: Type II

The product meets the requirements for Type II protective face masks according to the EN 14683:2019+AC:2019 standard.

TECHNICAL PROPERTIES

Bacterial filtration efficiency: ≥98%

Materials: meltblown non-woven PP, spunbond non-woven PP

Nose clip: PE-layered metal thread

Earloops: Polyester-Spandex compound, non-latex, flexible

USE OF MASK

The disposable medical face mask is designed to protect the people in the vicinity of the person using the mask. The mask prevents the possible exhaled pathogens from spreading.

In addition to mask use, persons are to follow other recommended safety measures such as the use of other protective gear, hand hygiene and social distancing. In professional use check if the medical face mask is appropriate for the task at hand.

The product is designed for both consumers and for professional use. The mask is disposable.

The device is not a personal protective equipment (PPE). If the user needs a personal protective equipment, they should acquire an EN 149 classified half mask.

The medical face masks have not been tested in potentially explosive atmospheres. The flammability of the face masks has not been tested. The face mask should not be used near open flames.

Material testing has not found any risk of skin irritation or injury to user. Some users (typically children, the mentally and physically handicapped or people afflicted with chronic respiratory diseases) may find the use of face masks difficult, and/or experience increased discomfort and difficulties in communication or breathing.

Report any serious incident that has occurred in relation to the device to the manufacturer and the local competent authority.

MAINTENANCE, CLEANING AND DISINFECTION

Medical face masks are disposable and should not be reused. If the mask is reused, there is an increased risk of contamination.







INSTRUCTIONS OF USE

First time users are to be instructed of the correct use of the masks. Failing to follow the guidelines in every phase of the mask use may lead to a health risk for the mask wearer.

CHECK BEFORE USE

Wash or disinfect your hands before touching the face mask. Check that the face-mask packaging is undamaged and that the mask is clean and intact. Inspect the use-by date, and do not use the face mask if it is too old.

Keep the face masks in their original packaging until they are used.

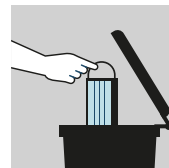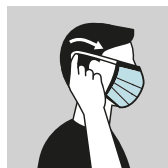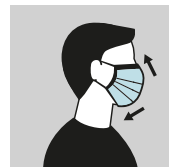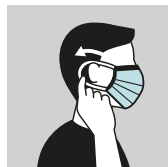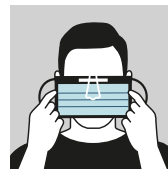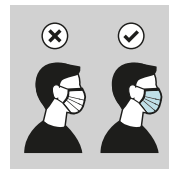THE USE, REMOVAL AND DISPOSAL OF THE PROTECTIVE DEVICE

Wash or disinfect your hands before using the mask. Place the face mask over your face by holding it by the earloops, the white side against the face. Fit the nose clip and the earloops carefully in place so that the mask covers the nose, mouth and chin tightly. Avoid touching the outer surface of the face mask during use. Use the mask continuously: do not remove it from the face temporarily to hang from below the chin for instance. If you are obliged to remove the face mask or it becomes damaged, dirty or moist, replace the mask with a new one.

Remember to wash or disinfect your hands before removing the face mask. Remove the mask by holding the earloops. Do not touch the outer surface of the protective gear. Dispose of the face mask by throwing it to a garbage bin directly after use. In professional use check the procedure for PPE waste disposal. Wash or disinfect your hands again after you have handled the face mask.

SYMBOL AND SIGN DESCRIPTIONS

Electronic Instruction for Use

Does not include natural rubber

Manufacturer

Do not re-use

Store out of direct sunlight

Batch code

Non-sterile

Use-by Date

Catalog number

Humidity saturation limit

Temperature limits

Store in a dry place

Medical Device

CE marking

Manufacturer

Lumi Dental Oy (2725049-9)

Satakunnankatu 7, 78300 Varkaus

info@lumimedical.com | Tel. 020 770 5420

This instruction was approved

2021-05-17.

IFU 11201-11502 v1

www.lumimedical.com